

Step-by-Step Guide for Solace Bed Extension Kit

Solace | Solace Deluxe | Solace Deluxe Low



Introduction to the Solace Bed Extension Kit

The Solace Bed Extension Kit is designed to enhance the versatility of your bed, making it suitable for taller residents. This kit is compatible with the Solace, Solace Deluxe and Solace Deluxe Low Profile Beds, ensuring a seamless extension for improved comfort and functionality.

To make installation even easier, we've included a QR code below. Simply scan it to watch a step-by-step video demonstration and follow along for quick and hassle-free setup.



■ Prepare the Bed for Extension

- Switch the bed to its lowest position and ensure it's disconnected from any power source for safety.
- Lock the bed's wheels to prevent movement during the installation.

Step 1: Tilt the Bed & Disconnect the Actuator Cable

Carefully tilt the Solace profile bed onto its side to allow access to the underside components. Locate the actuator cable connected to the footboard and disconnect it to ensure there is no power running to the footboard section during the installation.



Step 2: Remove the Thumbscrews from the Footboard

Locate the thumbscrews on both sides of the footboard. Unscrew and remove them, keeping them aside for reattachment.



Step 3: Remove the Footboard

Gently slide the footboard away from the bedframe and set it aside in a safe area.



Step 4: Insert the Bed Extension

Take the bed extension and slide it into the end of the bedframe where the footboard was removed. Ensure it is aligned properly with the frame.



Step 5: Secure the Extension

Secure the extension in place by tightening the screw and thumbscrew on both sides of the extension frame to hold it securely.



Step 6: Reattach the Footboard

Slide the footboard back onto the bedframe, aligning it with the extension.



Step 7: Secure the Footboard to the Extension

Use the screws and thumbscrews to secure the footboard onto the extension, tightening them on both sides to ensure a stable fit.



Step 8: Reconnect the Actuator Cable

Reconnect the actuator cable to the footboard to restore its functionality.



Step 9: Return the Bed to its Upright Position

Carefully tilt the bed back to its upright position and perform a final check to confirm that all components are securely attached and aligned.



Step 10: Reconnect Power and Test

Reconnect the power source and test all bed functions, including raising, lowering, and adjusting positions, to confirm stability and safety with the extension in place.



NHC Group Ltd, Unit 4, Didcot Way, Boldon Business Park, Boldon, NE35 9PD

T: (0)191 416 9512 E: info@nhcgroup.co.uk www.nhcgroup.co.uk